



METAL HALF STEP

K30 - METAL HALF STEP

K30L - LARGE METAL HALF STEP

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.



EN ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS) - CONTACT DETAILS OVER LEAF



SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:


EN




MAXIMUM USER WEIGHT

Your **LANGHAM METAL HALF STEPS** were specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

i K30

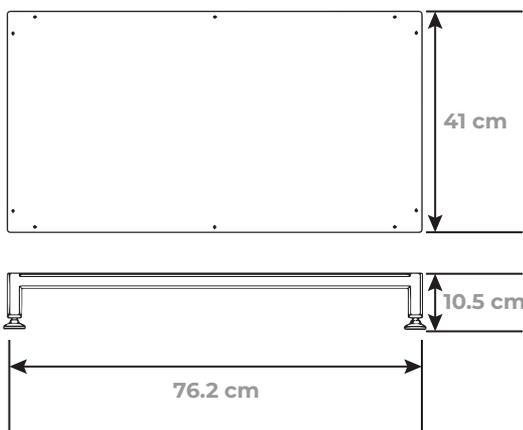

i K30L

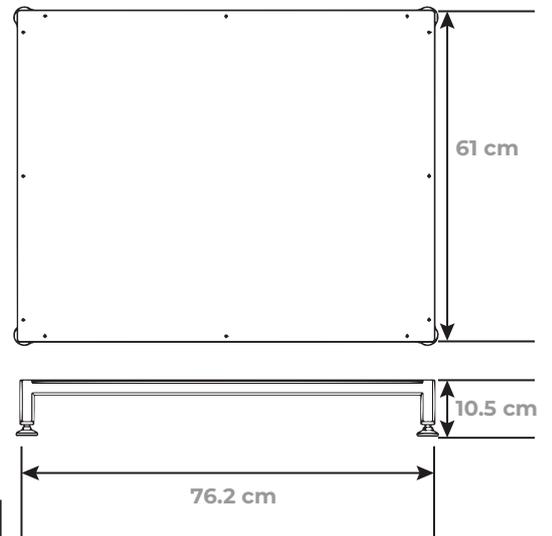

LANGHAM

INSPIRED BY YOUR NEEDS



MADE IN CHINA



GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA



MD

IFU-REF LP/K30/1020-1

PART REF LP/K30 + LP/K30L

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

gordonellis.healthcare
 @EllisHealthcare








INSTRUCTIONS FOR USE

i EN THE K30 METAL HALF STEP IS STRONG AND STEADY IN USE, WITH NYLON FEET THAT CAN BE ADJUSTED TO THE LEVEL OF THE GROUND. THE K30L HALF STEP INCLUDES A LARGER SURFACE, SUITABLE FOR A USER AND THEIR WALKING FRAME.

1. SCREW ONE NUT ONTO THE THREAD OF EACH FOOT AND INSERT THE FEET INTO THE CORNERS OF THE STEP.
2. ENSURE THE STEP IS LEVEL BY ADJUSTING THE FEET AS NECESSARY. WHEN LEVELLED TIGHTEN THE NUT AGAINST THE UNDERSIDE OF THE STEP USING THE SPANNER PROVIDED. DO NOT SCREW THE FEET OUT TOO FAR - A MAXIMUM LENGTH OF 38MM FROM THE BASE OF THE LEG TO THE BOTTOM OF THE FOOT IS ALLOWED.
3. POSITION THE STEP OUTSIDE OF THE BUILDING ENTRANCE IN THE ORIENTATION SHOWN. THE LONGEST SIDE OF THE STEP MUST BE ADJACENT TO THE ENTRANCE OF THE BUILDING.
4. TEST FOR STABILITY.
5. STAND CENTRALLY ONTO THE STEP WITH BOTH FEET BEFORE STEPPING OFF.

A SUITABLY PLACED GRAB BAR/RAIL WILL ASSIST ALL USERS OF THE HALF STEP TO ACHIEVE A MORE SECURE TRANSFER.

! DEVICE IS FOR STATIC USE ONLY ON A LEVEL SURFACE. IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. CHECK FOR BUILD UP OF ICE OR SURFACE WATER BEFORE USE. CHILD AND TODDLER USE UNDER SUPERVISION ONLY, NEVER LEAVE ALONE. TAKE CARE IF KNOCKED DURING USE. THE STEP IS DESIGNED WITH A NON-SLIP SURFACE - IT IS NOT RECOMMENDED TO COVER THE STEP WITH A MAT. FEET ARE FOR LEVELLING PURPOSES ONLY - NOT HEIGHT ADJUSTMENT. PERIODICALLY CHECK FIT OF FEET AND RETIGHTEN IF NECESSARY. DEVICE NOT FOR ANY OTHER USE. DO NOT USE IN CONJUNCTION WITH OTHER HALF STEPS.

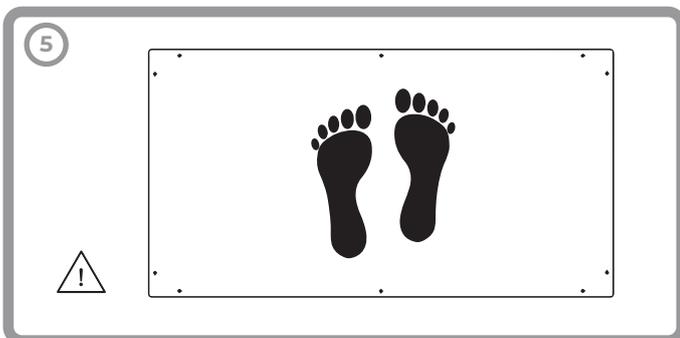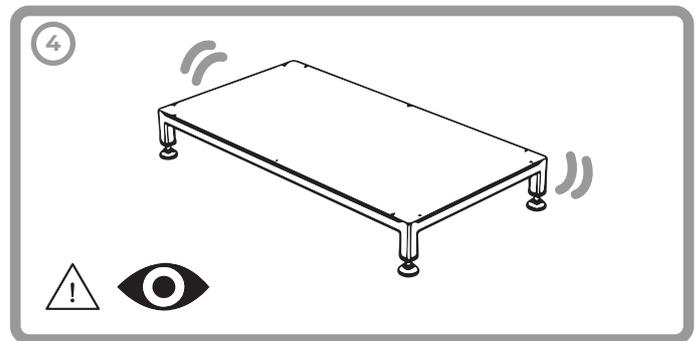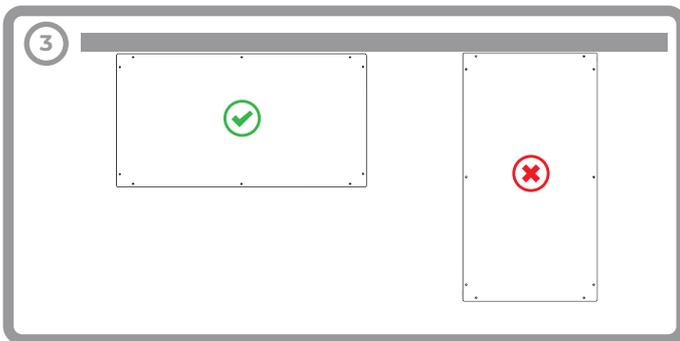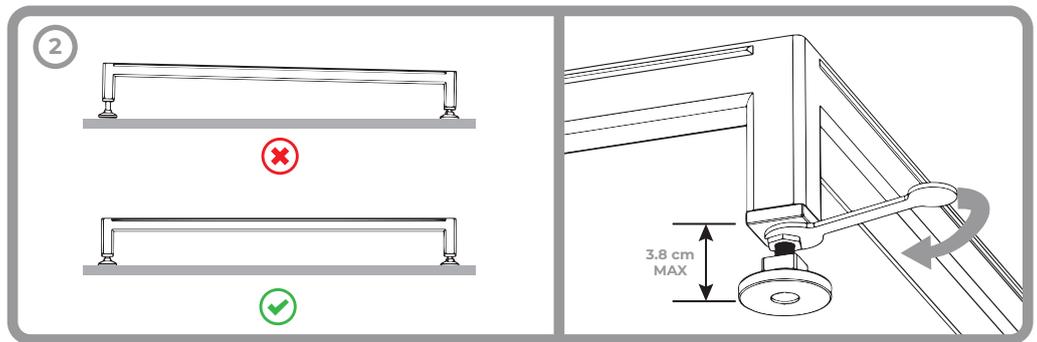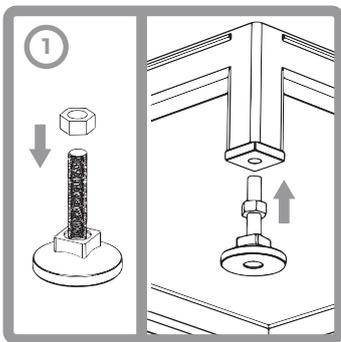

i CLEANING INSTRUCTIONS



WIPE DOWN WITH A
NON-ABRASIVE CLOTH
AND SOAPY WATER

PH 5 - 9

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

 [gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)
 [@EllisHealthcare](https://twitter.com/EllisHealthcare)

**UK
CA** 