

# INSTRUCTIONS FOR USE

**EN** THE SUPPORT HANDLE IS DESIGNED TO ASSIST PEOPLE WHO ARE UNSTEADY ON THEIR FEET TO ACHIEVE A MORE SECURE TRANSFER USING THE STEP. IF REQUIRED A SUPPORT HANDLE CAN BE FITTED TO BOTH SIDES OF THE STEP. IT IS TO BE USED ONLY WHEN FITTED TO A LANGHAM METAL HALF STEP (K30) OR LANGHAM LARGE METAL HALF STEP (K30L).

PLEASE REFER TO THE INSTALLATION GUIDE SHOWN INSIDE.

## TO FIT FLOOR FIXING AND HANDLE:

1. USING THE LARGE SPANNER, REMOVE THE FEET FROM THE SIDE OF THE STEP THE HANDLE IS TO BE FITTED.
2. FIT FLOOR FIXING BRACKET TIGHTENING THE TOP NUT TO SECURE THE BRACKET USING THE LARGE SPANNER.
3. ALIGN AND FIT THE HANDLE BRACKET AS SHOWN. SCREW THE FOOT THROUGH THE BRACKET AND RETIGHTEN THE NUT USING THE LARGE SPANNER TO CLAMP THE BRACKET AND LOCK THE FOOT IN PLACE.
4. INSERT THE BOLT ON THE BACK SIDE OF THE BRACKET, ON THE INSIDE OF THE STEP. TIGHTEN THE NUT USING THE SMALL SPANNER AND HEX KEY. ENSURE THE BRACKET IS SECURELY IN PLACE.
5. REPEAT PREVIOUS STEPS ON OTHER LEG.
- 6 + 7. ALIGN HANDLE WITH THE REMAINING HOLES ON BRACKET. INSERT THE BOLTS THROUGH THE HOLES AND TIGHTEN THE NUTS USING THE SMALL SPANNER AND HEX KEY.

## TO FIT FLOOR FIXING ONLY:

1. USING THE LARGE SPANNER, REMOVE THE FEET FROM THE SIDE OF THE STEP THE HANDLE IS TO BE FITTED.
2. FIT FLOOR FIXING BRACKET TIGHTENING THE TOP NUT TO SECURE THE BRACKET USING THE LARGE SPANNER.
3. REINSERT THE LEG INTO THE STEP.
4. RETIGHTEN THE NUT AGAINST THE FRAME OF THE STEP TO LOCK THE FOOT IN PLACE

INSTALL THE SECURITY FIXING KIT ON ALL FEET; DO NOT USE THE SUPPORT HANDLE WITHOUT INSTALLING THE SECURITY FIXING KIT. ENSURE HANDLE AND STEP ARE SECURE AND STABLE BEFORE USE. TREAD CENTRALLY ONTO THE STEP SURFACE AVOIDING STEPPING ON TO THE CORNERS OR ALONG THE EDGES OF THE STEP, DO NOT LEAN BACKWARDS AS THIS IS LESS STABLE FOR THE USER. IT IS THE RESPONSIBILITY OF THE OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. PERIODICALLY CHECK ALL FIXINGS FOR TIGHTNESS. CHILD AND TODDLER USE UNDER ADULT SUPERVISION ONLY. HANDLES ARE FOR SUPPORT ONLY AND ARE NOT LOAD BEARING TO THE SAME WEIGHT AS THE STEP.



PH 5 - 9

## CLEANING INSTRUCTIONS

**EN** WIPE DOWN WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.



# METAL HALF STEP

K30H - METAL HALF STEP HANDLE

K30LH - LARGE METAL HALF STEP HANDLE

Welcome to your new Gordon Ellis product.  
We hope that it proves a useful aid in your daily life.



**SEE INSIDE AND REVERSE FOR FITTING INSTRUCTIONS**  
AVAILABLE IN THE FOLLOWING LANGUAGES:  
EN



**HANDLES FOR SUPPORT ONLY.**




**EN** ONLY TO BE USED WITH THE LANGHAM METAL HALF STEP (K30) OR LANGHAM LARGE METAL HALF STEP (K30L) - PLEASE READ THESE INSTRUCTIONS ALONG WITH THE STEP INSTRUCTIONS BEFORE USE.


ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS) - CONTACT DETAILS BELOW

Your **LANGHAM METAL HALF STEP HANDLE** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. With the aim of making daily living that little bit easier.

**K30H**

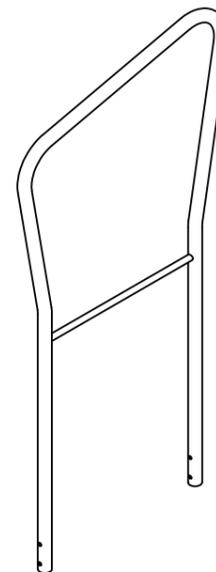

**K30LH**

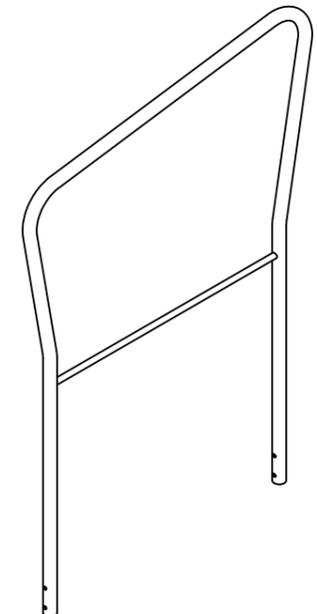

DESIGNED IN THE UK

MADE IN CHINA



**GORDON ELLIS & CO.**  
TRENT LANE  
CASTLE DONINGTON  
DERBY, ENGLAND DE74 2AT

EC REP

**AUTHORISED REPRESENTATIVE**  
ADVENA LTD, TOWER BUSINESS CENTRE,  
2ND FLOOR, TOWER STREET, SWATAR, BKR  
4013 MALTA

PART REF LP/K30H + LP/K30LH

IFU-REF LP/K30H/0122-4

VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)

[@EllisHealthcare](https://twitter.com/EllisHealthcare)



VIEW THE ENTIRE RANGE ONLINE AT  
[GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK)

[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)

[@EllisHealthcare](https://twitter.com/EllisHealthcare)












# SUPPORT HANDLE AND FLOOR FIXING

Tools Required: 1 x Dual Sided Spanner







1 x 4mm HEX KEY

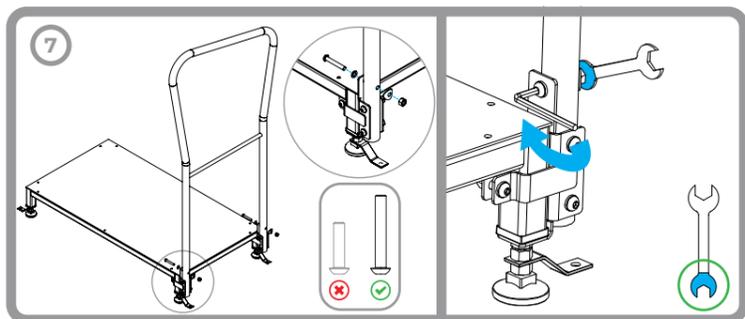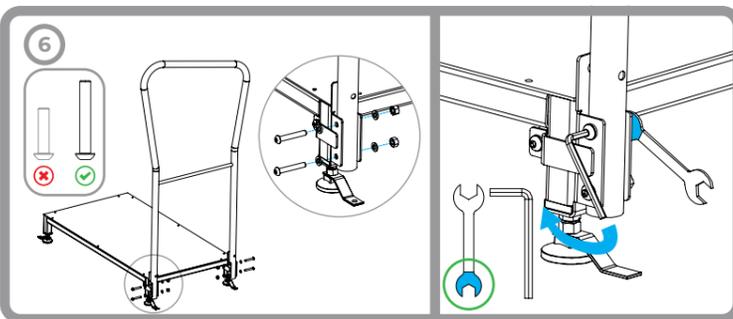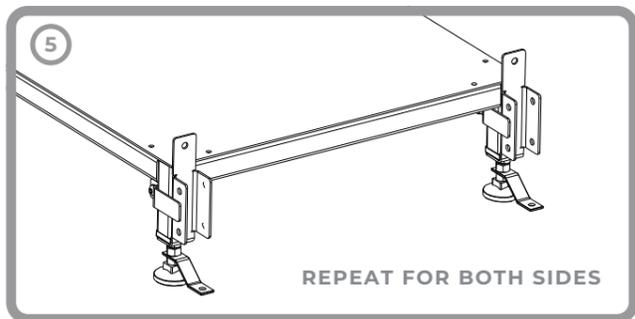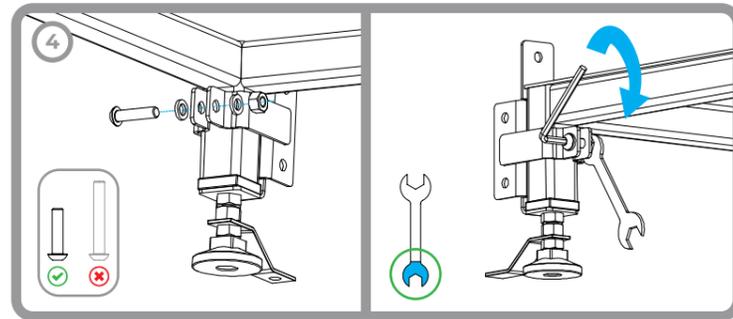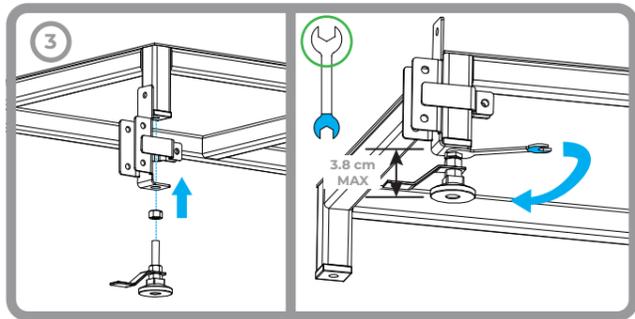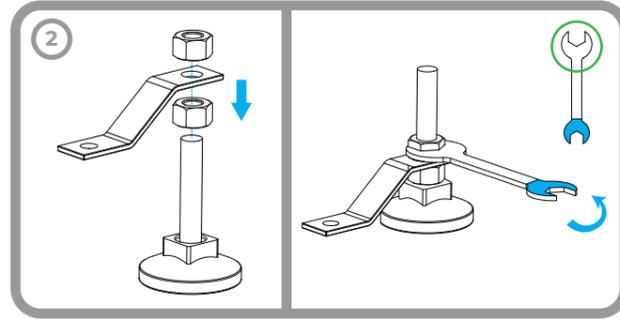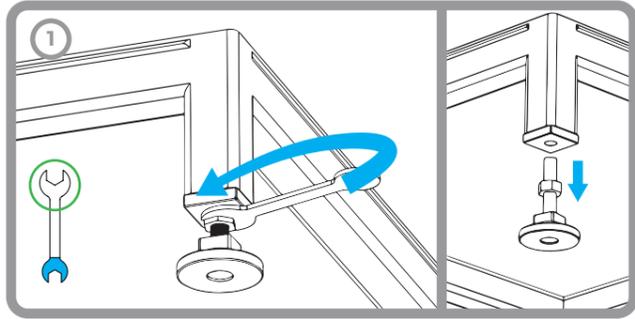

# FLOOR FIXING ONLY

Tools Required: 1 x Dual Sided Spanner

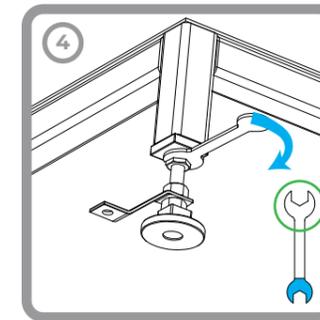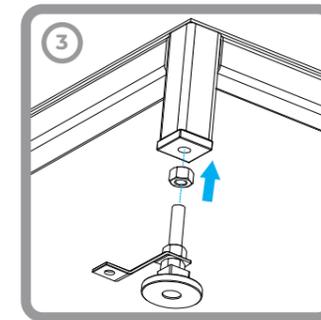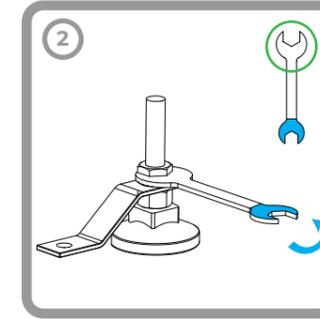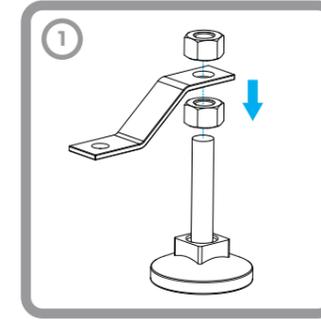

## INSTRUCTIONS FOR USE



**EN** ENSURE STEP IS LEVEL. ALL LEGS SHOULD BE INSERTED TO THE SAME HEIGHT. AT LEAST 1.5CM OF THREAD MUST REMAIN INSIDE THE FRAME AT ALL TIMES FOR SAFE USE. ENSURE STEP IS STABLE BEFORE EACH USE.

