

ASHBY SLOPED RAISED TOILET SEAT

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily living.

Your **ASHBY SLOPED RAISED TOILET SEAT** was specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.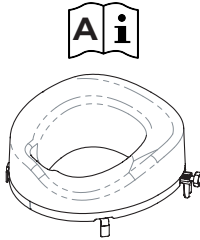

**ASHBY LHS
SLOPED TOILET SEAT
15CM RAISE**
66274

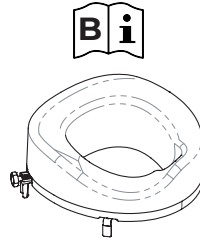**ASHBY RHS
SLOPED TOILET SEAT
15CM RAISE**
66284

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY, ENGLAND DE74 2AT

EC REP

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

MD

IFU-REF 66274/0424-2

REF 66274 + 66284

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK




[gordonellis.healthcare](https://www.facebook.com/gordonellis.healthcare)

@EllisHealthcare







ASSEMBLY INSTRUCTIONS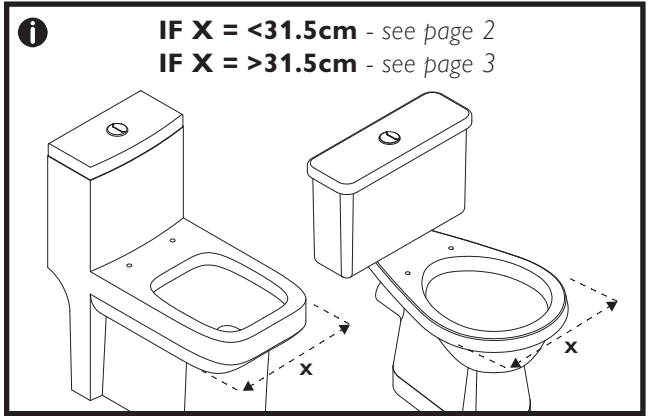
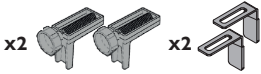
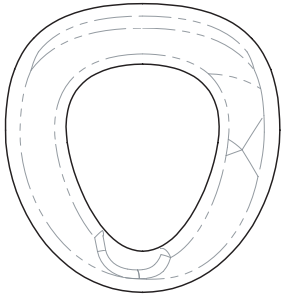


IF X = <31.5cm

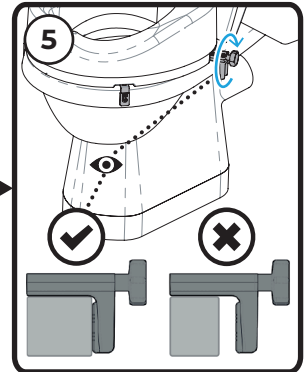
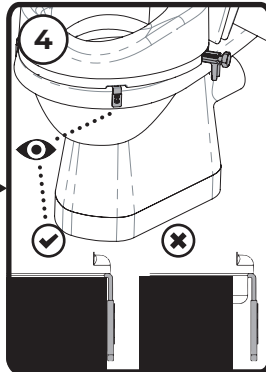
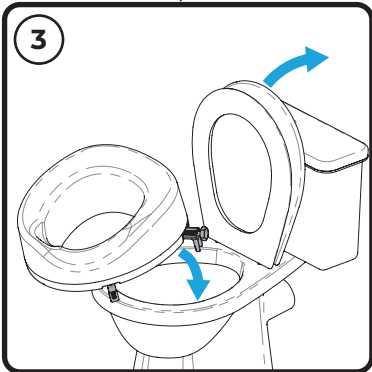
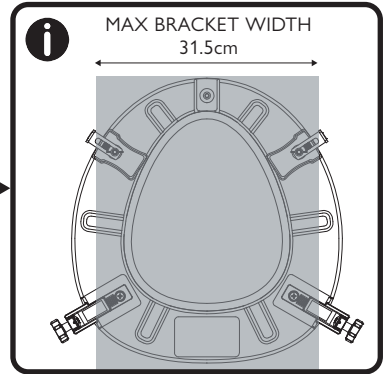
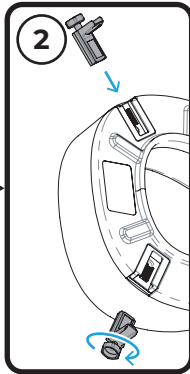
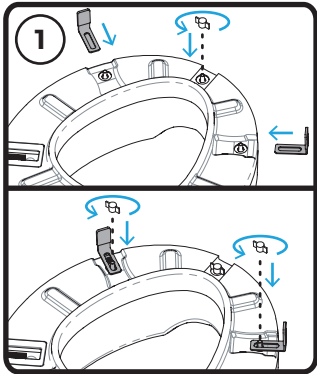
ASSEMBLY INSTRUCTIONS

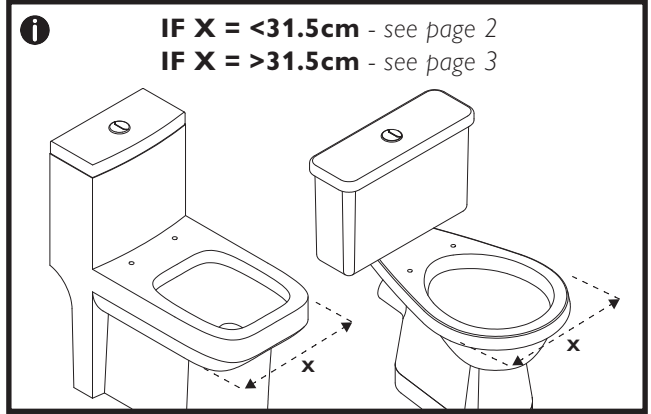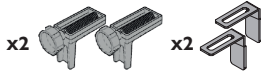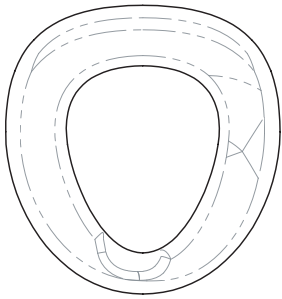IF X = >31.5cm

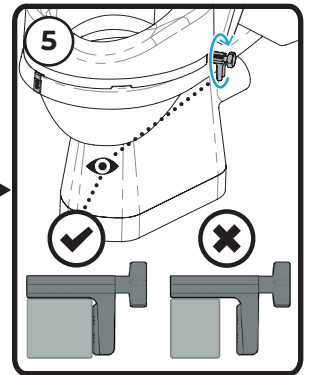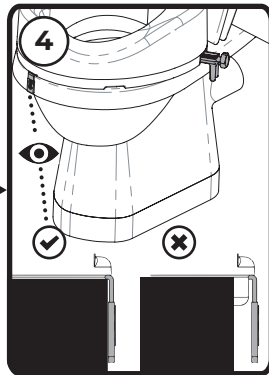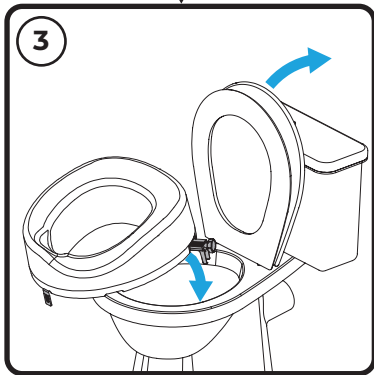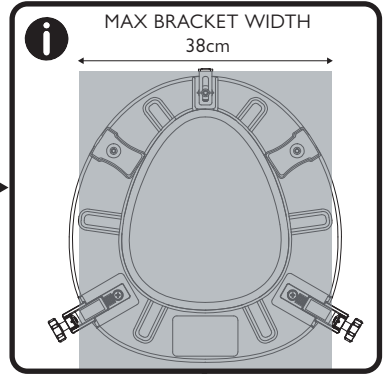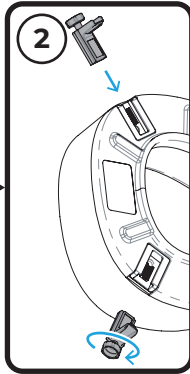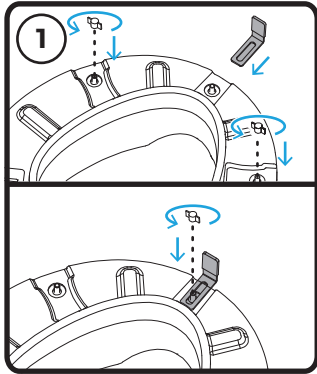EN

ONLY USE ASHBY SLOPED RAISED TOILET SEAT AS SHOWN. CHECK THE SEAT IS SECURE BEFORE EACH USE. BRACKETS SHOULD BE ADJUSTED TO GIVE A TIGHT FIT. PERIODICALLY CHECK FIT IS STILL TIGHT. AVOID OVER TIGHTENING BRACKETS TO STOP SEAT FROM RIDING UP.

SELECT THE APPROPRIATE ASSEMBLY METHOD BASED ON THE TOILET WIDTH. THE MAXIMUM WIDTH FOR THE 4 BRACKET ASSEMBLY IS 31.5CM. THE MAXIMUM WIDTH FOR THE 3 BRACKET ASSEMBLY IS 38CM. WHERE POSSIBLE THE 4 BRACKET ASSEMBLY IS RECOMMENDED FOR MAXIMUM STABILITY.

† = MAXIMUM USER WEIGHT.

Δ = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER (UK & NON-EU CUSTOMERS)

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