
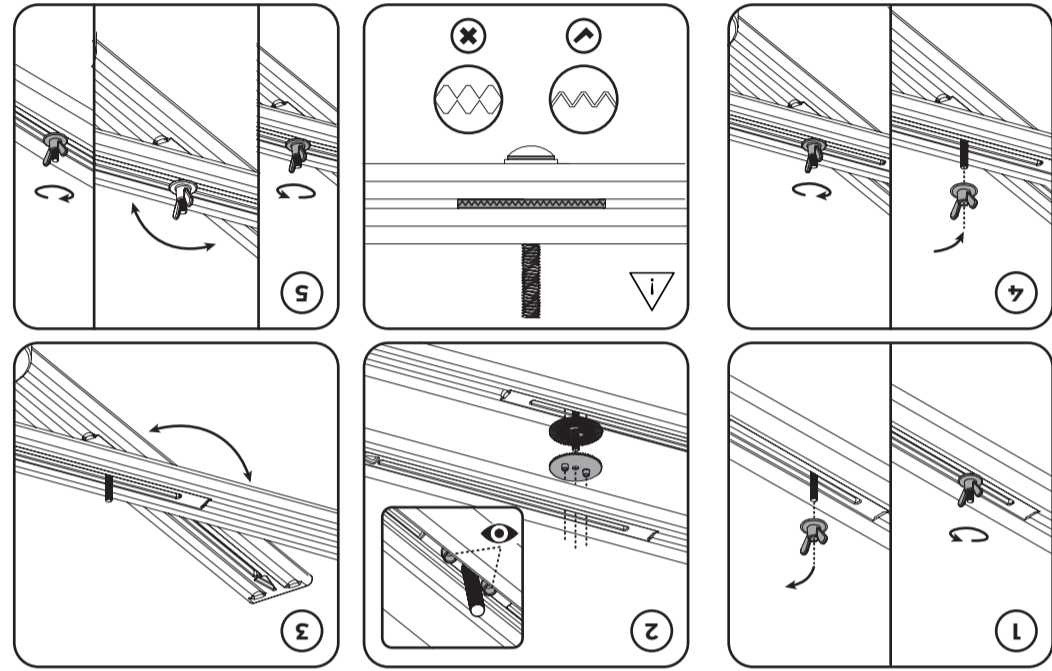
**NLP ASSEMBLY INSTRUCTIONS:** 1. POSITION UNITS ON THE FLOOR PARALLEL TO THE FRONT AND REAR OF BED. 2. LOOSEN THE WING NUTS. 3. ADJUST TO SIZE SO THAT BED LEGS WILL DROP INTO THE RAISER UNITS. LIFT THE HEAVIEST SIDE OF THE BED FIRST. TYPICALLY THE HEAD END AND PLACE LEGS INTO SOCKETS. 4. LIFT THE FOOT END AND PLACE LEGS INTO SOCKETS. ENSURE THE RAILS RUN SIDE TO SIDE TO MINIMIZE THE RISK OF A TRIP HAZARD. 5. PUSH RAISER UNITS TIGHT UP AGAINST THE CORNERS OF THE LEGS. 6. TIGHTEN THE WING NUTS. 7. RAISER BLOCKS CAN BE INSERTED AS NECESSARY TO FURTHER RAISE BY 24MM OR 48MM - MAXIMUM TWO PER RAISER UNIT. ENSURE EACH RAISER UNIT IS EQUAL IF EXTRA HEIGHT IS ADDED. 8. ENSURE THE BED AND RAISER UNITS ARE STABLE BEFORE EACH USE AND IF KNOCKED.

**SURELOCK INSTRUCTIONS:** 1. FULLY UNSCREW WING NUT AND SET ASIDE. 2. INSERT THE TWO SURELOCK WASHERS WITH THE TWO TOOTHED SURFACES FACING EACH OTHER. ENSURE THE BOSSES CORRECTLY SIT WITHIN THE RAIL ADJUSTMENT SLOTS. 3. ALIGN AND POSITION THE FURNITURE AS REQUIRED. 4. WHEN IN THE DESIRED POSITION REATTACH THE WING NUT AND TIGHTEN. (I) WHEN POSITIONING THE RAISERS ENSURE THE TEETH OF THE LOWER SURELOCK SIT WITHIN THE TEETH OF THE UPPER SURELOCK. MISALIGNED TEETH OR TEETH NOT PROPERLY SEATED WILL CAUSE THE "LOCK" TO FAIL. 5. ENSURE THE WING NUT IS LOOSENED PRIOR TO ADJUSTING OR REPOSITIONING THE FURNITURE RAISER. THE WING NUT DOES NOT NEED TO BE FULLY REMOVED. THE WING NUT MUST BE RETIGHTENED FOLLOWING ANY ADJUSTMENTS.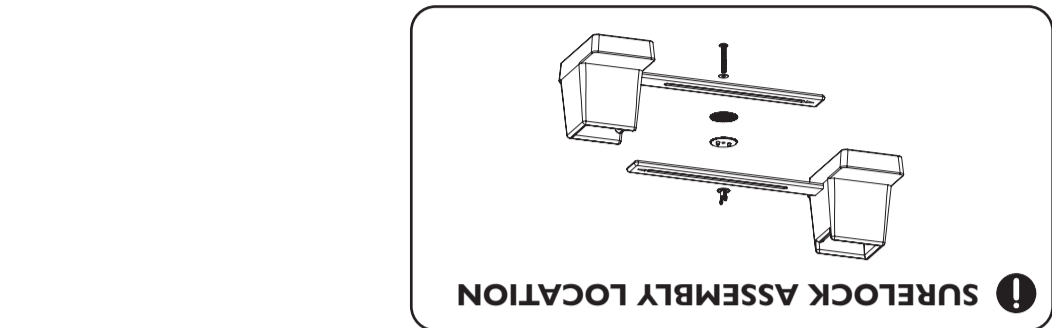

**SURELOCK INSTRUCTIONS - OPTIONAL**

ALWAYS REMOVE TRAPED SURELOCK WASHERS FROM THE RAILS BEFORE USE OF THE RAISER. IF USING THE WASHER FOLLOW THE STEPS BELOW. ALTERNATIVELY, SET ASIDE AS USUALLY. THE RAISER AS NORMAL.

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







**FR**

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Triman - Indicate recycling and sorting rules

**FR**

NOTICE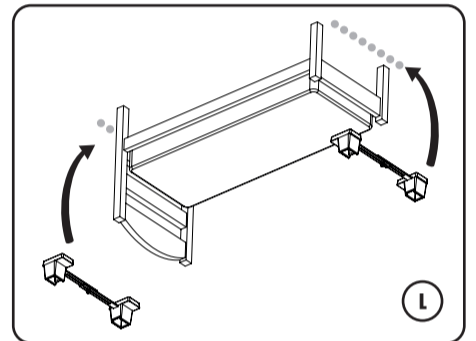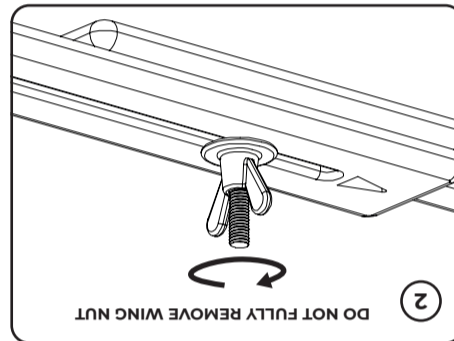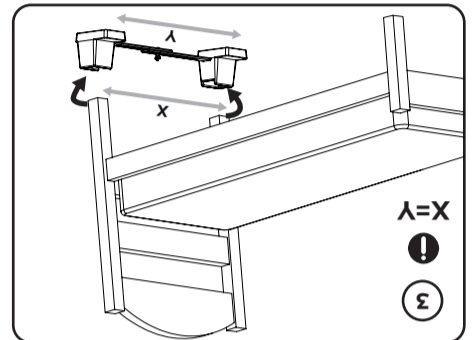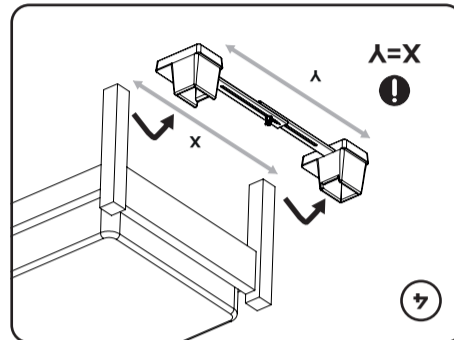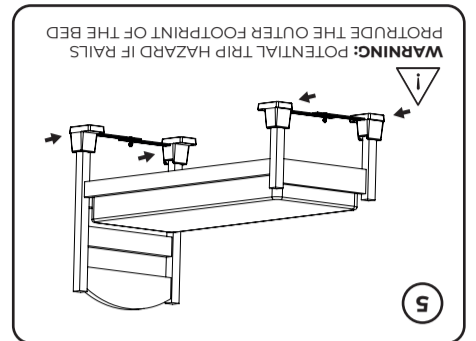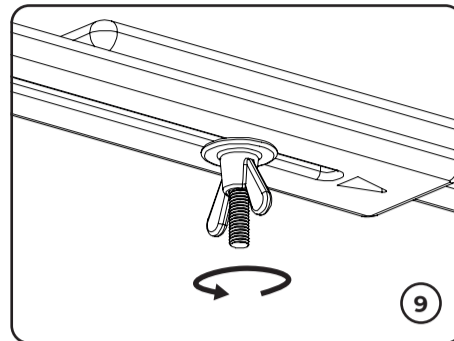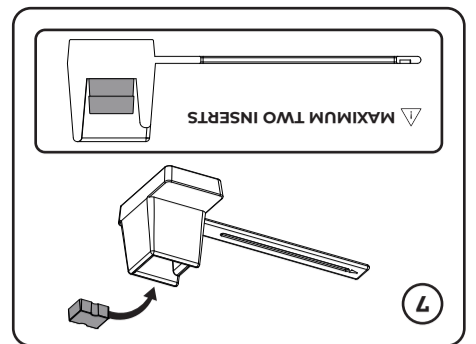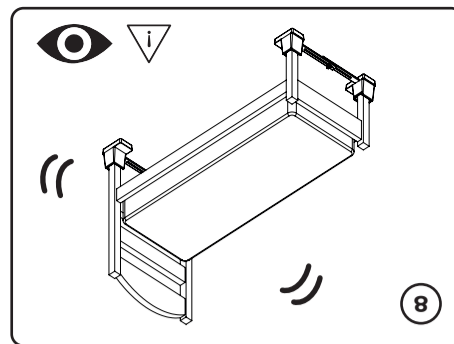


**NLP ASSEMBLY INSTRUCTIONS**

# LANGHAM ADJUSTABLE BED RAISER


Welcome to your new Gordon Ellis product. We hope that it proves a useful aid in your daily life.

Your **LANGHAM ADJUSTABLE BED RAISER** was specifically developed by our UK Design Team who have over 60 years experience in raising furniture, coupled with feedback received from Occupational Therapists. Designed for beds with supporting legs, with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

- LP/NLP129**  
SUITABLE FOR: DOUBLE BEDS  
SUITABLE BED WIDTHS: 91.4 - 146cm (36 - 57.5 INCHES)  
MAXIMUM BED LEG SIZE: 6.4 X 7.6cm (2.5 - 3 INCHES)
- LP/NLP129S**  
SUITABLE FOR: SINGLE BEDS  
SUITABLE BED WIDTHS: 61 - 89cm (24 - 35 INCHES)  
MAXIMUM BED LEG SIZE: 6.4 X 7.6cm (2.5 - 3 INCHES)

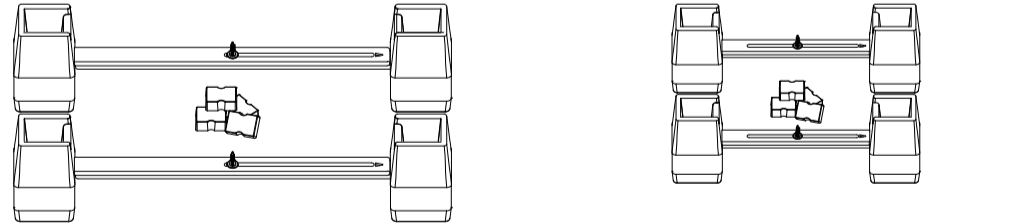**RAISING HEIGHT**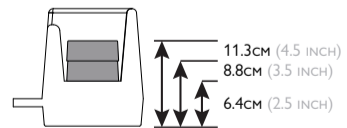

DESIGNED & MADE IN THE UK | BHTA | 5 PP | GORDON ELLIS & CO. TRENT LANE, CASTLE DONINGTON, DERBY, ENGLAND DE74 2AT | EC REP | AUTHORIZED REPRESENTATIVE ADVENA LTD, TOWER BUSINESS CENTRE, 2ND FLOOR, TOWER STREET, SWATAR, BKR 4013 MALTA

VIEW THE ENTIRE RANGE ONLINE AT [GORDONELLISHEALTHCARE.CO.UK](http://GORDONELLISHEALTHCARE.CO.UK) | [gordonellis.healthcare](http://gordonellis.healthcare) | UK CE